

SAFETY AND MOBILITY

CareWatch
Fall Prevention System

TAMPER RESISTANT ALARM WITH NURSE CALL INTERFACE

USER MANUAL

OPERATING INSTRUCTIONS:

1. Remove tamper-resistant screw on battery compartment cover on front of alarm. Remove battery cover. Insert three "AAA" batteries included in package.

PLEASE NOTE: Alarm will immediately sound once batteries are installed. To shut alarm sound off press firmly on red dot under "Reset" 3 times within 1.5 seconds. Alarm will immediately stop sounding. Slide battery compartment cover back in place. Tighten tamper-resistant screw. Make certain cover is securely in place.

2. Loosen locking security screw and slide patient-proof cover off alarm. Using end of a pen:
 - a. Set Volume lever (1) to desired decibel level. The range is from 70-90 dB.
 - b. Set Tempo lever (2) to desired tempo from quick to slow.
 - c. Set Tone lever (3) to desired tone.
3. To use nurse call interface slide nurse call wire down into open slot at the end of alarm and plug into opening marked "Nurse Call Jack." Tuck "Pull" tabs into compartment.
4. Slide connecting wire of pad through open slot at end of alarm and plug into opening marked "Pad Jack." Tuck "Pull" tabs into compartment. Slide nurse call and pad connecting wires up through open slot at end of alarm. To remove jacks pull up on tabs marked "Pull."



CAUTION! ALWAYS remove sensor pad pin from alarm by pulling straight up on the white "PULL" tab. DO NOT attempt to remove pin any other way or you may damage pin and void warranty.

5. Slide alarm's tamper-resistant cover back into place and tighten locking security screw.
6. If using Alarm holder, place alarm into holder connecting Velcro piece on alarm's back to Velcro piece sewn on back of holder. Affix holder to an appropriate place out of reach of resident. If not using holder affix alarm to bed or night stand using Velcro strip enclosed in package.
7. If using the nurse call interface connect the standard 1/4" nurse call wall jack into wall outlet for your nurse call system. Check to make certain your nurse call system is functioning properly.
8. To test alarm for battery and proper function press the red dot next to "Test" in the upper left hand corner of the alarm. Alarm will sound 4 times to indicate that it is set to function properly. You can stop the alarm from sounding anytime by pressing firmly on the red dot under "Reset" 3 times within 1.5 seconds. When user returns to pad the alarm will automatically reset.
9. Place resident so that resident's buttocks are in center of bed or chair pad. When resident is placed on pad alarm will automatically be set to alarm if patient attempts to get out of bed or chair. To stop alarm from sounding once activated press firmly on the red dot under "Reset" 3 times within 1.5 seconds. When user returns to pad the alarm will automatically reset.
10. The red "Low Battery Indicator" light will come on when it is time to change the batteries.

PLEASE NOTE: THIS ALARM SYSTEM MUST BE TESTED EACH TIME IT IS USED!

CAUTION! This product is not a restraint. It will not prevent a resident from getting out of their bed or chair. This product is designed as a monitor, alerting the staff when the resident has risen from their bed or chair. This product is to be used in conjunction with a total fall prevention program.