



# LEARN IN PERSON AGAIN

## COVID-19 SELF-TESTING HELPS MAKE IT POSSIBLE

OUR RAPID TESTING TECHNOLOGY  
CAN HELP KEEP SCHOOLS SAFE.

Educators, parents, and students are ready for a new normal—one that allows them to return to class with confidence and peace of mind. The Panbio™ COVID-19 Antigen Self-Test may be supplied for self-testing in the home or other environment.








FREQUENT, RAPID COVID-19 SELF-TESTING CAN HELP GIVE STUDENTS, PARENTS, AND EDUCATORS THE PEACE OF MIND TO REOPEN SCHOOLS, MINIMISING DISRUPTION TO EDUCATION.

## PANBIO™ COVID-19 ANTIGEN SELF-TEST

- Simple, fast and reliable — easy even for people who have never tested themselves.
- Leverages technology and established performance of professional use test.
- Performance data specifically focused on children.
- Trusted brand used in more than 100 countries



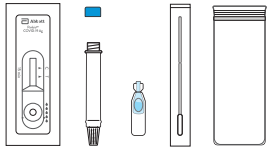
# DETECTION OF ACTIVE COVID-19 INFECTION WITH OR WITHOUT SYMPTOMS IN ONLY 15 MINUTES

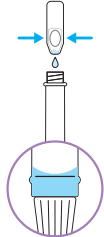
-  Illustrated step-by-step instructions
-  Simple test procedure using a nasal swab
-  Results in 15 minutes; no need to wait for a healthcare provider
-  Can be used in all ages. Children under 14 years old should be supported by an adult.
-  Data on performance in individuals with and without symptoms

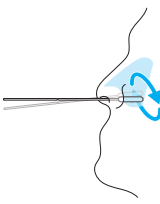



## EASY-TO-USE, SIMPLE PROCEDURE WITH COMFORTABLE NASAL SWAB

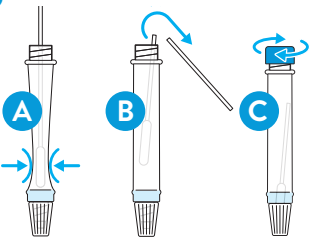
Please read the full Instructions for Use that come in the product kit box before testing. These instructions are the test procedure to complete a single test. Kit component quantities may vary.

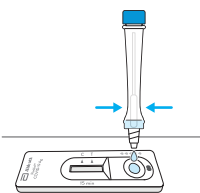
- 


PREPARE MATERIALS\*
- 

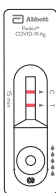
FILL TUBE
- 

SWAB NOSE
- 

SWIRL SAMPLE
- 

PINCH TUBE (A), BREAK OFF SWAB STICK (B), PLACE BLUE CAP ON TUBE (C)
- 

ADD 5 DROPS
- 

WAIT 15 MINUTES
- 

READ RESULT

### NOTE

Negative test results do not exclude infection with SARS-CoV-2 (COVID-19).

Positive test results or symptomatic persons require immediate confirmatory testing with a polymerase chain reaction (PCR) test.

# INDIVIDUALLY PACKED WITH INSTRUCTIONS FOR SELF-TESTING

## 1 TEST PACK

CATALOG NUMBER:

41FK51

KIT CONTENTS:

- 1 Instructions for Use
- 1 Test Device
- 1 Tube
- 1 Blue Cap
- 1 Buffer Bottle
- 1 Swab
- 1 Disposal Bag
- 1 Tube Rack



## 4 TEST PACK

CATALOG NUMBER:

41FK91

KIT CONTENTS:

- 1 Instructions for Use
- 4 Test Devices
- 4 Tubes
- 4 Blue Caps
- 4 Buffer Bottles
- 4 Swabs
- 4 Disposal Bags
- 1 Tube Rack



## 10 TEST PACK

CATALOG NUMBER:

41FK71

KIT CONTENTS:

- 2 Instructions for Use
- 10 Test Devices
- 10 Tubes
- 10 Blue Caps
- 10 Buffer Bottles
- 10 Swabs
- 10 Disposal Bags
- 1 Tube Rack



## 20 TEST PACK

CATALOG NUMBER:

41FK81

KIT CONTENTS:

- 4 Instructions for Use
- 20 Test Devices
- 20 Tubes
- 20 Blue Caps
- 20 Buffer Bottles
- 20 Swabs
- 20 Disposal Bags
- 2 Tube Racks



CONTACT YOUR LOCAL REPRESENTATIVE TODAY.

VISIT [SELFTEST.ABBOTT](https://selftest.abbott)

T: 1800 622 642 (free call AU) | T: 0800 466 951 (free call NZ) | T: +61 7 3363 7100 | [rapiddx.ANZ.enquiries@abbott.com](mailto:rapiddx.ANZ.enquiries@abbott.com)

Product not available in all countries. Not available for sale or distribution in New Zealand. Consult your local Abbott representative for availability in your country.

© 2021 Abbott. All rights reserved. All trademarks referenced are trademarks of either the Abbott group of companies or their respective owners. Any photos displayed are for illustrative purposes only. COL-06847-02 11/21

