

CAREOUSEL




Automatic Pill Dispenser (Model 3/07) Instructions



SAFETY AND MOBILITY PTY LTD

www.safetyandmobility.com.au

CAREOUSEL AUTOMATIC PILL DISPENSER MODEL 3/07

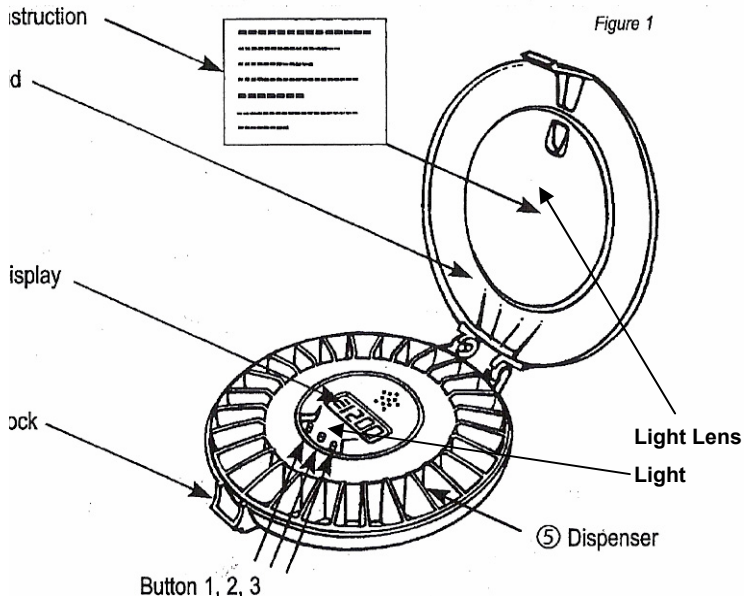



The pack also includes:-

6. Medication
booklet and
case

7. Three reversible
day/time discs

8. Batteries
4 x AA Alkaline

9. Key

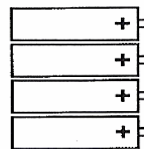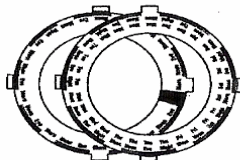

CAREOUSEL AUTOMATIC PILL DISPENSER MODEL 3/07

The Careousel Automatic Pill Dispenser reminds the user by means of an alarm and flashing light when to take their medication. It will make the correct dose available at the correct time of day or night, whilst keeping other pills locked out of sight. Medication is dispensed from an internal tray containing 29 sections.

Maximum number of daily alarm settings - 28.

When the alarm sounds the internal tray will rotate one position. The correct dosage will come into view in the lid opening. Pills should be removed from the dispenser by tilting it and pouring the pills into the hand or suitable container

This will also cancel the alarm signal and the flashing light.

Technical data:

Dimensions	- Length 190mm Width 180mm Height 56mm
Weight	- 380g excl. batteries 480g incl. batteries
Compartment volume	- Approx 10ml per compartment
Power supply	- 6V (4 x 1.5V Alkaline batteries)
Battery life	- > 1 year

USING THE DISPENSER FOR THE FIRST TIME

Open the lid as illustrated.

Peel backing from instruction label (1) and stick on the inside of the lid.

Open the lid to the battery box (underneath the dispenser) and activate the dispenser by removing the battery isolating strip.

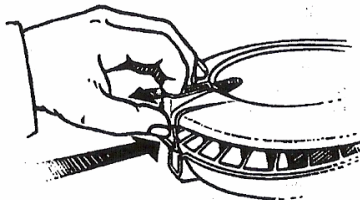


Figure 2

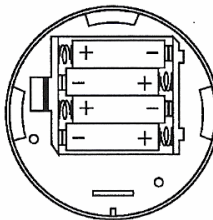


Figure 3

FILLING THE DISPENSER

1 Time Discs

Three day / time discs are supplied which comprise:

Two printed discs for 1 up to 4 medication prompts per day (use either side) and one blank disc for more than 4 medication prompts per day.

The discs should be completed by hand indicating the medication times.

NB: Before filling out the day/time discs, calculate which day/time you expect the refilled trays to start (particularly if pharmacies are refilling the trays) and start to the right of the red section.

Place the appropriate disc (7) for the dosage in question on top of the tray (5).

Secure the disc in place by bending down the four tabs - (a spot or two of glue will secure it further if preferred)

2. Medication (also see note on back page under 'Medication')

Remove the tray from the dispenser.

Starting from the right of the solid red section (not used) place the correct dosage in each section of the tray.

After filling, replace the tray in the dispenser so that the empty compartment marked by the solid red section will be visible through the lid opening – figure 4.

Ensure that the tray is placed in the dispenser so that when the lid is closed the right side of the pill compartment is flush with the right side of the lid opening.

At the next alarm time the tray will rotate one section clockwise and make available the correct medication.

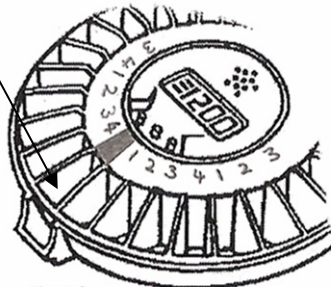


Figure 4

3. Spare trays with lids and time discs can be purchased to allow pre-filling.

4. Complete the medication booklet, place inside the case, slide the case under the dispenser as shown and clip into position. (figure 5)

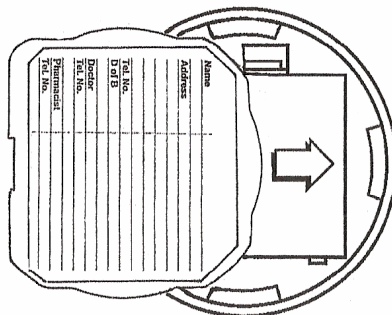


Figure 5

DISPENSER SETTINGS

There are two setting menus :-

Menu A:- Button 1
DOSES
ALARM
CLOCK

Menu B:- Button 3
BEFORE
SOUND
HOURS
RECORD
AFTER
LIGHT
DATE
TEST

Note:

After completion of settings in either Menu A or B press button 1 for three seconds to restore to normal operation. If nothing is done the display will automatically return to normal after 30 seconds.

After completion of settings we recommend the dispenser is tested (see TEST function on page 7. following)

Menu A (accessed through Button 1)

Set CLOCK:

1. Press Button 1 repeatedly until "CLOCK" is visible on the display.
2. Set the correct time by pressing Buttons 2 or 3. You can fast forward or reverse the time by pressing and holding down these buttons.

Set DOSES per day:

1. Press Button 1 until "DOSES" is visible on the display.
2. Wait a few seconds. Adjust using Buttons 2 and 3 to the correct number of doses per day.

Set ALARM times:

1. Press Button 1 repeatedly until "ALARM" is visible on the display.
2. Set the 1st (1) alarm time using buttons 2 or 3.
3. Press Button 1 again for the 2nd (2) alarm time and repeat the procedure for all subsequent alarm times required.

MENU B (Accessed through Button 3)

BEFORE

Taking medication before the pre-set alarm time

It is possible to take medication earlier than at the normal pre-set alarm time:

Turn the dispenser upside down and then turn upright again. Do this twice. The dispenser will then rotate one step and the next dose will become available.

(Cont'd on page 6 overleaf)

BEFORE (Continued)

The time in advance that this can be done is controlled by the 'BEFORE' function, to set this:

Press and hold Button 3 until BEFORE is displayed.

Press Buttons 2 or 3 to set the time required (0-30 minutes maximum)

NB: If medication is taken before the normal time, the dispenser will not activate again until the following medication is due.

If this function is not required set at 00.00 .

AFTER

The length of time the alarm will sound / light will flash can be adjusted by the 'AFTER' function: Press and hold Button 3 until BEFORE is displayed.

Press Button 1 repeatedly until 'AFTER' is displayed.

Select the required alarm duration by pressing Buttons 2 and 3 (minimum 5 minutes / maximum 1 hour)

SOUND

There are four alarm sounds available. To choose the alarm sound:

Press and hold Button 3 until BEFORE is displayed.

Press Button 1 repeatedly until "SOUNDS" is displayed.

Choose your preferred sound 01 02 03 or 04 by pressing Buttons 2 or 3.

Should no sound be required set at 00.

The alarm will sound every 4 seconds for the first minute, and every 30 seconds thereafter until medication is taken or the alarm duration limit is reached (see 'AFTER' function below)

LIGHT

Press and hold Button 3 until BEFORE is displayed. Press Button 1 repeatedly until 'LIGHT' is displayed.

If the light is not required press button 2 once – '00' will be displayed'

If the light is required press button 3 once – '01' will be displayed.

HOURS

24hr / AM-PM clock settings

The dispenser is preset to display the time as a 12 hour AM/PM display.

Should you wish to change this to 24 hour clock setting:

Press and hold Button 3 until BEFORE is displayed.

Press Button 1 repeatedly until "HOURS" is displayed.

Choose am/pm or 24hrs by pressing buttons 2 or 3.

DATE (for automatic BST / GMT adjustment)

Press and hold Button 3 until BEFORE is displayed. Press Button 1 repeatedly until 'DATE' is displayed.

'YY' is displayed – press Buttons 2 and 3 to set the year then press 1

'MM' is displayed – press Buttons 2 and 3 to set the month then press 1

'DD' is displayed – press Button 2 and 3 to set the date then press 1

RECORD

The dispenser will record the actual times the previous 28 medications were dispensed.

Press Button 3 for three seconds, then Button 1 until 'RECORD' is displayed – the time that the last medication was taken is displayed. Press Buttons 2 or 3 for the previous medication times to be shown.

TEST

The dispenser has an inbuilt test function.

To test the rotation and alarm sounds press button 3 for a few seconds.

Press Button 1 repeatedly until 'TEST' is displayed.

Press Button 3 again – the tray will rotate, 'MOVING' is displayed and then 'TAKE' is displayed and the alarm will sound. Tilt the dispenser to stop the alarm. 'TEST' is then displayed. Hold Button 1 down to return to clock.

Note: If on **first use** the dispenser does not rotate and the alarm does not sound, remove a battery, replace and test again. The dispenser will then operate normally.

To test the '**BEFORE**' function, press button 3 for a few seconds.

Press Button 1 until 'TEST' is displayed.


Press Button 2. Close the lid and then turn the dispenser upside down twice. Turn back to the upright position.

The tray will rotate and the alarm will sound. Tilt the dispenser again to cancel the alarm.

LOCK

The dispenser can be locked with the key supplied. Turn the key a quarter of a turn clockwise to lock, anti-clockwise to unlock.

Please note that the lock is a simple device intended as an additional barrier only.

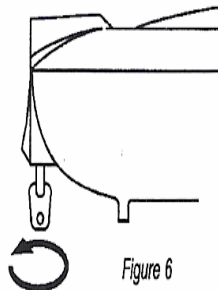

EXTERNAL INTERFACE

The dispenser includes an external interface to enable connection of various existing and future support electronic devices, for example - wireless transmitters, modems, alarm sounders, alarm lights etc.

For further information please contact Careousel.

GENERAL INFORMATION

Medication:

There are some pills that should not be exposed to the air and therefore should not be used in the dispenser. If in doubt please check with your pharmacist.

Fault handling:

The alarm will sound continuously in the event of the dispenser failing to rotate properly. This might be caused by an obstruction between the body of the dispenser and the internal tray. In this event the display will show "ERROR". A battery should be removed and then replaced. Any obstruction should be removed.

If the dispenser is upside down when the alarm occurs, the internal tray will not move until the dispenser is turned upright.

If the dispenser is upside down for all the time between the 'BEFORE' and 'AFTER' settings a warning signal will sound and 'TILT' will be displayed. Press Button 3 once to cancel. Check the position of the internal tray to ensure that the correct medication will be available at the next alarm time.

Batteries:

The batteries will last for one year in normal use.

When the battery voltage is reduced to approx. 5.2 volts, low battery warning "LOWBATT" is displayed. The batteries should be replaced as soon as possible.

When the battery voltage is reduced to approx. 4.8 volts, "ERROR" is displayed.

The dispenser will no longer function.

Remember when changing batteries to insert them to correct way around (see figure 3 – Page 3)

Cleaning: Dispenser - wipe with a damp cloth. **Internal tray** – may be washed in warm water or a dish washer (max. 35C)

Transport: If for any reason the dispenser is despatched by Australia post, courier or other carrier, it is important that **all batteries** are removed from the dispenser.

IMPORTANT

TO AVOID DAMAGING THE MECHANISM THE PILL TRAY MUST NOT BE MOVED AROUND MANUALLY WHEN INSIDE THE CASE. TO REPOSITION THE TRAY ALWAYS LIFT IT OUT AND REPLACE BY LOWERING DIRECTLY INTO THE CASE IN THE CORRECT POSITION.

Careousel will accept no responsibility for faults or any other consequences arising from misuse e.g. dropping, water damage, insertion of foreign objects, forced turning of the pill tray or forced opening of the dispenser

Safety and Mobility Pty Ltd
Tel: +61 (02) 9983 9520 Fax: +61 (02) 9983 9546
headoffice@safetyandmobility.com.au