

How to use your Cleaning Kit

- 

1 pull the bite valve off the tube
- 

2 insert the cleaning brush into the tube
- 

3 push all the way through twisting as you go
- 

4 clean the bite valve using the brush to reach inside
- 

5 reattach the bite valve to the tube

THE HYDRANT™

Proven to reduce hospital admissions

Lack of easy, independent access to fluids is a major cause of dehydration and its associated problems.


*“Simply Brilliant.
Brilliantly Simple”*

Sir Ranulph Fiennes BT OBE



THE HYDRANT™

Also available from Hydrate for Health

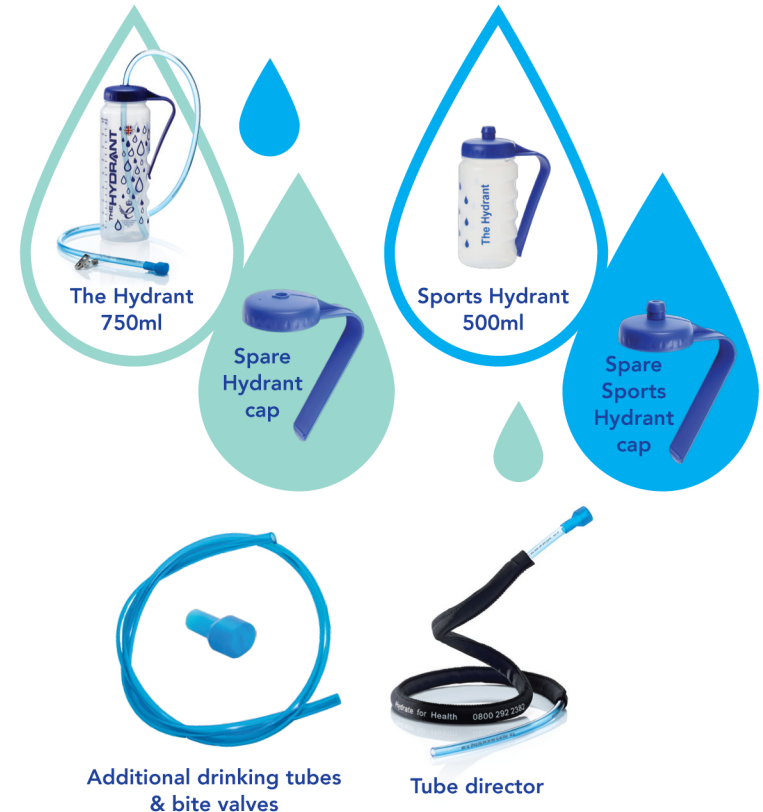

See our website for full details and pricing
www.alphamedicalsolutions.com.au

Get in touch

E: Info@alphamedicalsolutions.com.au

P: 1300 783 747 - ABN 17 434 257 512

PO Box 745 - St Ives - NSW - 2075



THE QUEEN'S AWARDS
FOR ENTERPRISE
INNOVATION 2013

