



BATH BOARD (SERIES: RBN)

GENERAL CARE & USE

The bath board spans the width of the bath and rests on the sides; it is used to enter and exit a bath.

SAFETY

- **Before using, be sure your weight is within the weight capacity of the unit.**
- These instructions are guidelines only. Use only as instructed by your healthcare professional.
- The Bath Board can be slippery when wet - use this product with caution.
- Product is intended to be used as a sitting surface only - do not stand on the Bath Board.
- Do not tip or put an offset load onto the Bath Board or injury may occur.
- Inspect frequently to ensure all screws are tight.
- Clean with water and mild soap.
- Temperature range: 3 to 70° C

WARNING

- Do not install the bath board if there is a shower door track on the side of the bath.
- Ensure the edge of the bath board against the wall is as close to the wall as possible.
- The side of the bath against the wall must be wide enough for the bath board to rest safely on it (minimum 25mm to 50mm).
- It is not recommended using the bath board on free standing baths, or baths with no side wall.
- When installed, the bath board should not exceed the outside wall of the bath by more than 20mm.
- Supervision or assistance is recommended for users with limited mobility when using the bath board.
- Ensure the bath board is securely in place before use.
- Using a slip proof bath mat increases safety, when using the bath board.

INSTALLATION

1. As shown in Figure 1:

Secure the handle (4) to the bath board (1) using handle screws (5)

2. Figure 2: Ensure there is a minimum horizontal flat surface of 38mm on the tub rim for the Bath Board overhang to rest on as shown. Adjust friction feet to fit snugly inside bathtub (a).

Tighten the screw to secure friction feet in place (b)

Note: the side of the Bath Board with the handle MUST be closest to the wall

Figure 1

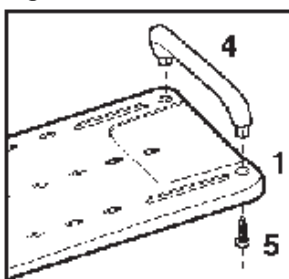


Figure 2

