


Mini-Spike® 2

Instructions For Use

B | BRAUN
SHARING EXPERTISE

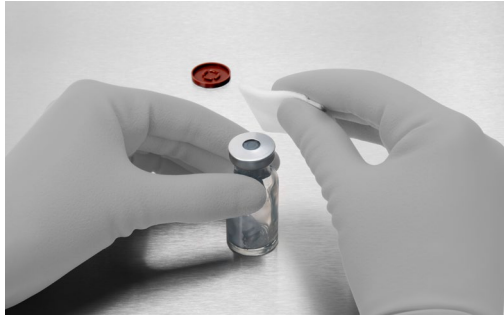

Remove protective cap from vial, disinfect stopper.



Remove the protective cap from the spike and pierce it into the vial by holding the grip plate on the Mini-Spike® 2. Make sure not to touch the female Luer lock connector on the Safeflow valve when doing so.



Disinfect the female Luer lock connector with a moistened disinfection cloth or the SwabCap® disinfection cap.



Connect to the syringe to the female Luer lock connector in the standing vial.



Flip the vial over and withdraw the solution.



To remove air from the syringe, withdraw half of the total desired quantity and flick the side of the filled syringe. Press the plunger on the syringe to expel the air from the syringe, and then withdraw the rest of the solution.



Disconnect the syringe from the female Luer lock connector on the upright vial and then withdraw the rest of the solution.



Disinfect the female Luer lock connector with a moistened disinfection cloth or the Swab-Cap® disinfection cap.



Instructional Video

B. Braun Australia Pty Ltd | Level 2, 4 Burbank Place, Norwest NSW 2153, Australia | Tel.1800 251 705 | info.au@bbraun.com | www.bbraun.com.au

E1124 | ©2024 B. Braun Australia™©2025 B. Braun Australia Pty Ltd. ABN 56 002 945 155. All rights reserved. Mini-Spike is a registered trademark of B. Braun Australia. This brochure is intended for use by healthcare professionals only. It is provided for informational purposes and does not constitute medical advice. Always refer to the product's Instructions for Use and consult a qualified healthcare provider for clinical decisions.